



My name is: _____


Intubation Date: _____

ET Tube Size: _____

Suction Catheter Size: _____

Circle the suction depth on the catheter picture for reference.

Colour bands allow easier visualization on subsequent suction procedures.

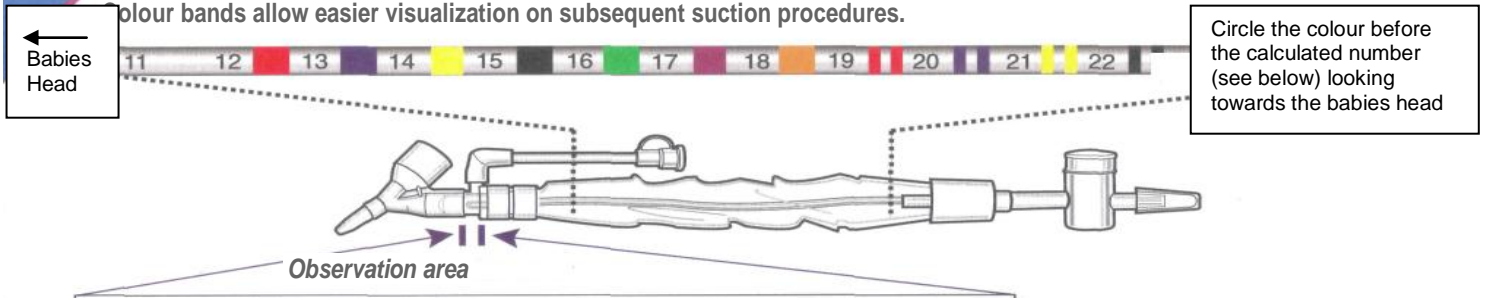

To Determine Suction Depth

(Observe in area directly across from lavage port connector)

Number on ET tube nearest endotracheal tube adapter tip	_____	cm
Add 5.0 cm to where ETT is cut	_____	cm
Equals suction depth:	_____	Total
		cm

KIMBERLY-CLARK*
Neonatal and Paediatric
TRACH CARE* Closed
Suction Systems